

# DETACHABLE ALUMINIUM COMMODE



## DETACHABLE ALUMINIUM COMMODE

### PRODUCT OVERVIEW

**KY692L**

**KJW-312L**



**KY692L**

**KJW-312L**

As a 3-in-1 Detachable Mobile Commode, a brilliant multi-tasker: a Commode Chair, a Shower Chair, as well as an Indoor Wheelchair.

- Foam Seat Cushions
- Lightweight Frame
- Detachable Bucket
- Detachable Foot Rest
- Flip Up Arm Rest
- 4 Wheels Lock

## DETACHABLE ALUMINIUM COMMODE

### PRODUCT FEATURES



#### LEG REST

Pull the catch release to remove detachable leg rest



#### ARM REST

Press button to release lock and flip up arm rest



#### LOCKABLE WHEELS

Push up to unlock and down to lock

### CARE INSTRUCTIONS

- When using mobility commode, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Commode.
- When getting in and out of the commode, please do not stand on the footplates.
- The footplates should be folded up beforehand, or removed.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the Commode.
- Do not exceed the maximum load.

## PRODUCT SPECIFICATIONS

	<b>KY692L</b>	<b>KJW-312L</b>
Overall Length	33.9" or 86cm	34.6" or 88cm
Overall Width	21.7" or 55cm	21.3" or 54cm
Overall Height	37.8" or 96cm	37.8" or 96cm
Seat Depth	17" or 43cm	18.1" or 46cm
Seat Width	17" or 43cm (Internal)	16.9" or 43cm (Internal)
Seat Height	18.5" or 47cm (Without Cushion) 20.9" or 53cm ( With Cushion)	19.6" or 50cm (Floor to Bucket Catch)
Height (Floor to Bucket Catch)	17" or 43cm	17" or 43cm
Front Wheel Diameter	5" or 12.5cm	5" or 12.5cm
Back Wheel Diameter	5" or 12.5cm	5" or 12.5cm
Safe Working Load	90 kg	90 kg
Overall Weight	10 kg	9 kg

Check our our full range of detachable commodes. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

