

STAINLESS STEEL COMMUNE



STAINLESS STEEL COMMODE

PRODUCT OVERVIEW

FS695S



FS697



- Flip Up Arm Rests
- Detachable Seats
- Detachable Buckets
- Detachable Foot Rests
- Foldable Foot Rests



FS695S





FS697

As a 3-in-1 Stainless Steel Mobile Commode, a brilliant multi-tasker: a Commode Chair, a Shower Chair, as well as an Indoor Wheelchair.

- PVC Leather Seat Cushions
- Lightweight Frame
- Detachable Bucket
- Detachable Foot Rest (FS697)
- Flip Up Foot Rest
- Flip Up Arm Rest
- 4 Wheels Lock

STAINLESS STEEL COMMODE

PRODUCT FEATURES

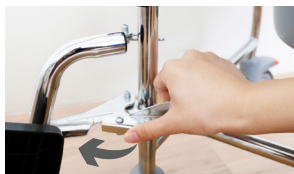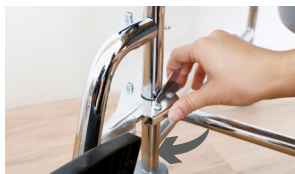

LEG REST - FS697

Pull the catch release to remove detachable leg rest



ARM REST - FS697

Press button to release lock and flip up arm rest



ARM REST - FS695S

Pull the black lever to release lock and flip up arm rest



LOCKABLE WHEELS

Push up to unlock and down to lock

CARE INSTRUCTIONS

- When using mobility commode, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Commode.
- When getting in and out of the commode, please do not stand on the footplates.
- The footplates should be folded up beforehand, or removed.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the Commode.
- Do not exceed the maximum load.

PRODUCT SPECIFICATIONS

	FS695S	FS697
Overall Length	35.4" or 90cm	2.3" or 82cm
Overall Width	21.3" or 54cm	21.3" or 54cm
Overall Height	37.8" or 96cm	38.6" or 98cm
Seat Depth	17" or 43cm	17.3" or 44cm
Seat Width	17.3" or 44cm	17.3" or 44cm
Seat Height	19.6" or 50cm	19.6" or 50cm
Height (Floor to Bucket Catch)	17" or 43cm	17" or 43cm
Front Wheel Diameter	5" or 12.5cm	5" or 12.5cm
Back Wheel Diameter	5" or 12.5cm	5" or 12.5cm
Safe Working Load	110 kg	100 kg
Overall Weight	12 kg	11 kg

Check our our full range of detachable commodes. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

