

# INOVYS II EVO RECLINING WHEELCHAIR



*For illustration purposes only*

## INOVYS II EVO RECLINING WHEELCHAIR

### PRODUCT OVERVIEW



### Inovys II Evo Reclining Wheelchair



- Smooth reclining backrest for exceptional comfort and posture support.
- Ergonomic design with padded seat, backrest, and leg supports for long-term comfort.
- Height-adjustable headrest for personalized neck and head positioning.

# RECLINING WHEELCHAIR

## PRODUCT FEATURES



### ADJUSTABLE FOOT REST

Flip up lever to adjust, flip down to lock.




### REMOVABLE LEG REST

Twist Lever to release leg rest, Pull up to remove.



### ADJUSTABLE ARM REST

Pull Knob and hold, adjust height and release. Pull all the way up to remove.

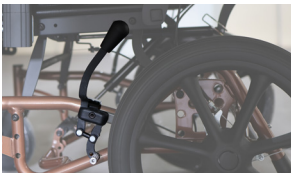

### STATIONARY HAND BRAKE

Push up to unlock and down to lock.



### HAND BRAKES

Press black lever to brake, press red lever to release.

# RECLINING WHEELCHAIR

## PRODUCT FEATURES



### HEAD REST ROTATE

Twist lever to right to release, Adjust and twist left to lock.



### ADJUSTABLE HEAD REST

Push lever down to release, push up to lock.



### REMOVABLE HEAD REST

Twist knob till removed, then pull head rest up.



### RECLINE

Squeeze upper handgrips and push down while user is sitting on wheelchair.

*For illustration purposes only*

## CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

## INOVYS II EVO RECLINING WHEELCHAIR

### PRODUCT SPECIFICATIONS

Overall Length	109 cm / 43"
Overall Width (Wheel to Wheel)	61.8 cm / 24.3"
Overall Height	122 cm / 48" - 108 cm / 42.5"
Seat Depth	47 cm / 18.5"
Seat Height	59.5 cm / 23.4"
Arm Rest Height (Floor to Arm Rest)	77 cm - 83 cm / 30.3" - 32.7"
Back Rest Height	63 cm / 24.8" - 48.5 cm / 19.1"
Reclining Angle	90° to 144°
Front Wheel Diameter	18 cm / 7"
Back Wheel Diameter	40.6 cm / 16"
Safe Working Load	135 kg
Overall Weight	32.5 kg
Designed in	Belgium
Country Of Origin	Made In China

Check our our full range of elevating wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

