

# BOBBY 12 LIGHTWEIGHT WHEELCHAIR



## BOBBY 12 LIGHTWEIGHT WHEELCHAIR

### PRODUCT OVERVIEW



#### Lightweight Wheelchair

- Easy to fold and operate
- Multi-function wheelchair with fold-up foot rests for easier patient transfer.
- Foldable back rest for easy storage
- Ventilated and cushioned seat upholstery
- Aluminium lightweight frame



*For illustration purposes only*

# LIGHTWEIGHT WHEELCHAIR

## PRODUCT FEATURES



### DETACHABLE ARM REST

Pinch and lift the lever to release the armrest lock, then pull upward to detach



### CAR BOOT FRIENDLY

Collapse wheelchair into a square to keep in your car boot



### FLIP UP FOOT REST

Flip up foot rest by pushing upwards

*For illustration purposes only*

## CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

## BOBBY 12 LIGHTWEIGHT WHEELCHAIR

### PRODUCT SPECIFICATIONS

Overall Length	94 cm or 37"
Overall Width (Wheel to Wheel)	57.5 cm or 22.6"
Overall Height	90.5 cm or 35.6"
Folded Length	76 cm or 29.9"
Folded Width (Wheel to Wheel)	33 cm or 13"
Folded Height	70.5 cm or 27.7"
Seat Depth	40 cm or 15.7"
Seat Width	47 cm or 18.5"
Seat Height (Floor to Seat Base)	50.5 cm or 19.5"
Arm Rest Height (Floor to Arm Rest)	70 cm or 27.6"
Back Rest Height	40 cm or 15.7"
Front Wheel Diameter	15 cm or 6"
Back Wheel Diameter	30 cm or 12"
Safe Working Load	115 kg
Overall Weight	12.3 kg

Check our our full range of lightweight wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

