

BOBBY ASTRO 12 LIGHTWEIGHT WHEELCHAIR



For illustration purposes only

VERMEIREN

BOBBY ASTRO 12 LIGHTWEIGHT WHEELCHAIR

PRODUCT OVERVIEW



Lightweight Wheelchair

- Easy to fold and operate
- Multi-function wheelchair with detachable foot rests for easier patient transfer.
- Foldable back rest for easy storage
- Ventilated and cushioned seat upholstery
- Aircraft grade aluminium lightweight frame
- Designed in Belgium

For illustration purposes only

VERMEIREN LIGHTWEIGHT WHEELCHAIR

PRODUCT FEATURES



FOLDABLE BACKREST

Press on the lever to release and fold back rest downwards for both sides



FLIP UP ARM REST

Press button to release lock and flip up arm rest



DETACHABLE FOOTREST

Pull the lever to release lock and flip up arm rest



STATIONARY HAND BRAKE

Push up to unlock and down to lock

For illustration purposes only

CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

BOBBY ASTRO 12 LIGHTWEIGHT WHEELCHAIR

PRODUCT SPECIFICATIONS

Overall Length	94 cm or 37"
Overall Width	58 cm or 22.8"
Overall Height	90 cm or 35.4"
Folded Length	68 cm or 26.8"
Folded Width	47 cm or 18.5"
Folded Height	70 cm or 27.6"
Seat Depth	40 cm or 15.7"
Seat Width	42 cm or 16.5"
Seat Height	52 cm or 20.5"
Arm Rest Height	70.5 cm or 27.8"
Back Rest Height	38 cm or 15"
Front Wheel Diameter	15.3 cm or 6"
Back Wheel Diameter	30.5 cm or 12"
Safe Working Load	115 kg
Overall Weight	12.3 kg

Check our our full range of lightweight wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

