ECL X1-16 ECLIPS LIGHTWEIGHT WHEELCHAIR



ECL X1-16 ECLIPS LIGHTWEIGHT WHEELCHAIR

PRODUCT OVERVIEW





Lightweight Wheelchair

- · Easy to fold and operate
- · Multi-function wheelchair for easier patient transfer.
- · Foldable back rest for easy storage
- · Ventilated and cushioned seat upholstery
- · Aircraft grade aluminium lightweight frame
- · Designed in Belgium



VERMEIREN LIGHTWEIGHT WHEELCHAIR

PRODUCT FEATURES





FOLDABLE BACKREST

Press on the lever to release and fold back rest downwards for both sides





STATIONARY HAND BRAKE

Push up to unlock and down to lock

For illustration purposes only

CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- · Lock the wheels before getting in and out of the Wheelchair.
- · When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- · Do not exceed the maximum safe working load.

ECL X1-16 ECLIPS LIGHTWEIGHT WHEELCHAIR

PRODUCT SPECIFICATIONS

Overall Length 96 cm or 37.8"

Overall Width (Wheel to Wheel) 64 cm or 25.2"

Overall Height 89.5 cm or 35.2"

Folded Length 96 cm or 37.8"

Folded Width (Wheel to Wheel) 31 cm or 12.2"

Folded Height 74 cm or 29.1"

Seat Depth 41 cm or 16.1"

Seat Width 45.5 cm or 18"

Seat Height (Floor to Seat Base) 47 cm or 18.5"

Arm Rest Height (Floor to Arm Rest) 67 cm or 26.4"

Back Rest Height 42 cm or 16.5"

Front Wheel Diameter 15.2 cm or 6"

Back Wheel Diameter 40.5 cm or 16"

Safe Working Load 90 kg

Overall Weight 13.2 kg

Check our our full range of detachable wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.



© 6223 0904 • 6931 7425 © 8501 1878 ≅ sales@rainbowcare.com.sg ⊕ rainbowcare.com.sg arainbowcare rainbowcaresg Blk 3007 Ubi Road 1 #02-410 Singapore 408701