

WHEELCHAIRS | USER GUIDE

rainbow care

ECL X1-16 ECLIPS LIGHTWEIGHT WHEELCHAIR



For illustration purposes only

VERMEIREN

ECL X1-16 ECLIPS LIGHTWEIGHT WHEELCHAIR

PRODUCT OVERVIEW



Lightweight Wheelchair

- Easy to fold and operate
- Multi-function wheelchair for easier patient transfer.
- Foldable back rest for easy storage
- Ventilated and cushioned seat upholstery
- Aircraft grade aluminium lightweight frame
- Designed in Belgium

VERMEIREN LIGHTWEIGHT WHEELCHAIR

PRODUCT FEATURES



FOLDABLE BACKREST

Press on the lever to release and fold back rest downwards for both sides



STATIONARY HAND BRAKE

Push up to unlock and down to lock

For illustration purposes only

CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

ECL X1-16 ECLIPS LIGHTWEIGHT WHEELCHAIR

PRODUCT SPECIFICATIONS

Overall Length	91.5 cm or 36"
Overall Width (Wheel to Wheel)	63.5 cm or 25"
Overall Height	91.5 cm or 36"
Folded Length	91.5 cm or 36"
Folded Width (Wheel to Wheel)	29 cm or 11.5"
Folded Height	89 cm or 35"
Seat Depth	40.5 cm or 16"
Seat Width	45.7 cm or 18"
Seat Height (Floor to Seat Base)	43 cm or 17"
Arm Rest Height (Floor to Arm Rest)	66 cm or 26"
Back Rest Height	40.5 cm or 16"
Front Wheel Diameter	15.2 cm or 6"
Back Wheel Diameter	40.5 cm or 16"
Safe Working Load	115 kg
Overall Weight	13.2 kg

Check our our full range of lightweight wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

