

# ECL X1-24 ECLIPS LIGHTWEIGHT WHEELCHAIR



*For illustration purposes only*

**VERMEIREN**

## ECL X1-24 ECLIPS LIGHTWEIGHT WHEELCHAIR

### PRODUCT OVERVIEW



### Lightweight Wheelchair



- Easy to fold and operate
- Multi-function wheelchair for easier patient transfer.
- Foldable back rest for easy storage
- Ventilated and cushioned seat upholstery
- Aircraft grade aluminium lightweight frame
- Designed in Belgium

## VERMEIREN LIGHTWEIGHT WHEELCHAIR

### PRODUCT FEATURES



#### **FOLDABLE BACKREST**

Press on the lever to release and fold back rest downwards for both sides



#### **STATIONARY HAND BRAKE**

Push up to unlock and down to lock

*For illustration purposes only*

### CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

## ECL X1-24 ECLIPS LIGHTWEIGHT WHEELCHAIR

### PRODUCT SPECIFICATIONS

|                                     |                   |
|-------------------------------------|-------------------|
| Overall Length                      | 105.5 cm or 41.5" |
| Overall Width (Wheel to Wheel)      | 68.5 cm or 27"    |
| Overall Height                      | 82 cm or 32.3"    |
| Folded Length                       | 105.5 cm or 41.5" |
| Folded Width (Wheel to Wheel)       | 33.5 cm or 13.2"  |
| Folded Height                       | 71 cm or 28"      |
| Seat Depth                          | 41 cm or 16.1"    |
| Seat Width                          | 46 cm or 18.1"    |
| Seat Height (Floor to Seat Base)    | 47 cm or 18.5"    |
| Arm Rest Height (Floor to Arm Rest) | 67.5 cm or 26.6"  |
| Back Rest Height                    | 42 cm or 16.5"    |
| Front Wheel Diameter                | 15.2 cm or 6"     |
| Back Wheel Diameter                 | 60.9 cm or 24"    |
| Safe Working Load                   | 90 kg             |
| Overall Weight                      | 14 kg             |

Check our our full range of detachable wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

