
WHEELCHAIRS | USER GUIDE

rainbow care

ECL X1-24 ECLIPS LIGHTWEIGHT WHEELCHAIR





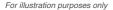

ECL X1-24 ECLIPS LIGHTWEIGHT WHEELCHAIR

PRODUCT OVERVIEW





Lightweight Wheelchair

- · Easy to fold and operate
- · Multi-function wheelchair for easier patient transfer.
- · Foldable back rest for easy storage
- · Ventilated and cushioned seat upholstery
- · Aircraft grade aluminium lightweight frame
- Designed in Belgium



VERMEIREN LIGHTWEIGHT WHEELCHAIR

PRODUCT FEATURES



FOLDABLE BACKREST

Press on the lever to release and fold back rest downwards for both sides

STATIONARY HAND BRAKE

Push up to unlock and down to lock

For illustration purposes only

CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- · Lock the wheels before getting in and out of the Wheelchair.
- · When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.



ECL X1-24 ECLIPS LIGHTWEIGHT WHEELCHAIR

Overall Length	102.8 cm or 40.5"
Overall Width (Wheel to Wheel)	68.5 cm or 27"
Overall Height	91.4 cm or 36"
Folded Length	102.9 cm or 40.5"
Folded Width (Wheel to Wheel)	36.8 cm or 14.5"
Folded Height	72.3 cm or 28.5"
Seat Depth	40.5 cm or 16"
Seat Width	45.7 cm or 18"
Seat Height (Floor to Seat Base)	45.7 cm or 18"
Arm Rest Height (Floor to Arm Rest)	67.3 cm or 26.5"
Back Rest Height	43.1 cm or 17"
Front Wheel Diameter	15.2 cm or 6"
Back Wheel Diameter	60.9 cm or 24"
Safe Working Load	115 kg
Overall Weight	14 kg

Check our our full range of lightweight wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.



€ 6223 0904 • 6931 7425 € 8501 1878 ≅ sales@rainbowcare.com.sg ⊕ rainbowcare.com.sg
I @rainbowcare @ rainbowcaresg Blk 3007 Ubi Road 1 #02-410 Singapore 408701