

WHEELCHAIRS | USER GUIDE

rainbow care

# ECL X2-24 ECLIPS DETACHABLE WHEELCHAIR



*For illustration purposes only*

**VERMEIREN**

## ECL X2-24 ECLIPS DETACHABLE WHEELCHAIR

### PRODUCT OVERVIEW



### ECL X2-24 Eclips Detachable Wheelchair

- Multi-function wheelchair with detachable foot rests and arm rests for easier patient transfer.
- Breathable woven nylon seat
- Lightweight aluminium frame

*For illustration purposes only*

## DETACHABLE WHEELCHAIR

### PRODUCT FEATURES



#### FOLDABLE BACKREST

Press on the lever to release and fold back rest downwards for both sides



#### FLIP UP ARM REST

Press button to release lock and flip up arm rest



#### DETACHABLE FOOTREST

Pull the lever to release lock and flip up arm rest



#### STATIONARY HAND BRAKE

Push up to unlock and down to lock

*For illustration purposes only*

### CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

## ECL X2-24 ECLIPS DETACHABLE WHEELCHAIR

### PRODUCT SPECIFICATIONS

Overall Length	99 cm or 38.9”
Overall Width (Wheel to Wheel)	66 cm or 25.9”
Overall Height	92 cm or 36.2”
Folded Length	80 cm or 31.4”
Folded Width	37 cm or 14.5”
Folded Height	76 cm or 29.9”
Seat Depth	41 cm or 16.1”
Seat Width	45 cm or 17.7”
Seat Height	46 cm or 18.1”
Arm Rest Height (Floor to Arm Rest)	67 cm or 26.3”
Back Rest Height	46 cm or 18.1”
Front Wheel Diameter	15 cm or 5.9”
Back Wheel Diameter	59 cm or 23.2”
Safe Working Load	120 kg
Overall Weight	15 kg

Check our our full range of detachable wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

