

# FS863-207-20 LIGHTWEIGHT WHEELCHAIR



*For illustration purposes only*

## FS863-207-20 LIGHTWEIGHT WHEELCHAIR

### PRODUCT OVERVIEW



### Lightweight Wheelchair

- Easy to fold and operate
- Multi-function wheelchair with flip up arm rest and foot rest for easier patient transfer
- Foldable back rest for easy storage
- Made with beathable woven nylon seat
- Comes with lightweight aluminium frame

# LIGHTWEIGHT WHEELCHAIR

## PRODUCT FEATURES



### FOLDABLE BACKREST

Press on the lever to release and fold back rest downwards for both sides



### STATIONARY HAND BRAKE

Push up to unlock and down to lock



### FLIP UP ARM REST

Press button to release lock and flip up arm rest

*For illustration purposes only*

## CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

## FS863-207-20 LIGHTWEIGHT WHEELCHAIR

### PRODUCT SPECIFICATIONS

Overall Length	97 cm or 38"
Overall Width (Wheel to Wheel)	62 cm or 24.5"
Overall Height	92 cm or 36.2"
Folded Length	84 cm or 33"
Folded Width (Wheel to Wheel)	31 cm or 12"
Folded Height	71 cm or 28"
Seat Depth	40 cm or 15.7"
Seat Width	46 cm or 18"
Seat Height (Floor to Seat Base)	50 cm or 20"
Arm Rest Height (Floor to Arm Rest)	67 cm or 26.5"
Back Rest Height	44 cm or 17"
Front Wheel Diameter	14 cm or 5.5"
Back Wheel Diameter	51 cm or 20"
Safe Working Load	90 kg
Overall Weight	12.5 kg

Check our our full range of lightweight wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

