

FS903 DETACHABLE ELEVATING WHEELCHAIR



FS903 DETACHABLE ELEVATING WHEELCHAIR

PRODUCT OVERVIEW



FS903 Detachable Elevating Wheelchair

- Multi-function wheelchair with detachable foot rests and flip up arm rests for easier patient transfer.
- Breathable woven nylon seat
- Lightweight aluminium frame

ELEVATING WHEELCHAIR

PRODUCT FEATURES



FOLDABLE BACKREST

Press on the lever to release and fold back rest downwards for both sides



FLIP UP ARM REST

Press button to release lock and flip up arm rest



ELEVATING FOOTREST

Pull the lever to release elevating leg rest



STATIONARY HAND BRAKE

Push up to unlock and down to lock

For illustration purposes only

CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

FS903 DETACHABLE ELEVATING WHEELCHAIR

PRODUCT SPECIFICATIONS

Overall Length	103 cm or 40.6"
Overall Width (Wheel to Wheel)	69 cm or 27"
Overall Height	93 cm or 36.5"
Folded Length	77 cm or 30.5"
Folded Width	37 cm or 14.5"
Folded Height	66 cm or 26"
Seat Depth	46 cm or 18"
Seat Height	44 cm or 17"
Arm Rest Height (Floor to Arm Rest)	66 cm or 26"
Back Rest Height	45 cm or 17.5"
Front Wheel Diameter	15 cm or 6"
Back Wheel Diameter	55.9 cm or 22"
Safe Working Load	95 kg
Overall Weight	15 kg

Check our our full range of elevating wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

