

BATHROOM AIDS | USER GUIDE

rainbow care

# STATIONARY COMMODE



*For illustration purposes only*

## STATIONARY COMMODE

### FT7511 STEEL PRODUCT OVERVIEW



### FT7511 PLASTIC PRODUCT OVERVIEW



#### Stationary Commode

Made of lightweight plastic frame and functions as a 2 in 1 commode where it is used as a commode chair and a shower chair. Rack for storage space.

## STATIONARY COMMODE

### PRODUCT FEATURES



#### HEIGHT ADJUSTMENT

Press the button and pull / push to the level that you want



#### REMOVE BUCKET

Pull the bucket from the front to remove the bucket

*For illustration purposes only*

## PRODUCT SPECIFICATIONS

FT7511

Overall Length 43 cm or 16.9"

Overall Width 53.5 cm or 21.1"

Overall Height 76 - 87 cm  
29.9" or 34.3"

Seat Depth 45.5 cm or 15.7"

Seat Width 40 cm or 15.7"

Seat Height 40.5 cm - 51.5 cm  
15.9" - 20.3"

Safe Working Load 110 kg

Overall Weight 5.5 kg

## CARE INSTRUCTIONS

- When using stationary commode, please pay attention to people and/or items around to avoid collision.
- Make sure the level of the height of each leg is the same before sitting down.
- Do not exceed the maximum load.

Check our our full range of stationary commodes. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

