FT6803 RECLINING ELEVATING WHEELCHAIR



FT6803 RECLINING ELEVATING WHEELCHAIR

PRODUCT OVERVIEW





FT6803 Reclining Elevating Wheelchair

Multi-function wheelchair with fliping away the foot rests and the flip up arm rests for easier patient transfer.

PVC Leather seat

Lightweight aluminium frame



RECLINING WHEELCHAIR

PRODUCT FEATURES





FOLDABLE BACKREST

Press on the lever to release and fold back rest downwards for both sides





FLIP UP ARM REST

Squeeze lever to release lock and flip up arm rest





FLIP-AWAY FOOTREST

Pull the lever to release lock and flip the foot rest





STATIONARY HAND BRAKE

Push up to unlock and down to lock

For illustration purposes only

CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

FT6803 RECLINING ELEVATING WHEELCHAIR

PRODUCT SPECIFICATIONS

Overall Length 116 cm or 45.6"

Overall Width (Wheel to Wheel) 66 cm or 26"

Overall Height 127 cm or 50"

Folded Length 78 cm or 30.7"

Folded Width 78 cm or 30.7"

Folded Height 73.5 cm or 28.9"

Seat Depth 45 cm or 17.7"

Seat Height 48 cm or 18.9"

Arm Rest Height (Floor to Arm Rest) 73 cm or 28.7"

Back Rest Height 80 cm or 31.5"

Front Wheel Diameter 15 cm or 5.9"

Back Wheel Diameter 50 cm or 19.7"

Safe Working Load 100 kg

Overall Weight 20 kg

Check our our full range of elevating wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

