# HS295U ELECTRICAL SCOOTER



## **HS295U ELECTRICAL SCOOTER**

### PRODUCT OVERVIEW



### **HS295U Electrical Scooter**



Electrical Scooter Scooter is a 4 wheeled electrical mobility scooter with a top speed of 8 km/h. With a width of 44.5cm, it is easy to maneuver through tight corridors and even within your home.

- Key Operated
- · Dynamic Controller
- · Electro-Mechanical Brake



### **ELECTRICAL WHEELCHAIR**

### PRODUCT FEATURES





# HANDLE ANGLE ADJUSTMENT

Pull the lever up to change the angle





### **SEAT DIRECTION**

Change the seat direction by pushing down the lever while turning



For illustration purposes only

### **CARE INSTRUCTIONS**

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels or turn off the motor before getting in and out of the Electrical Wheelchair.
- Do try to charge the Electrical Wheelchair with direct wall plug.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- · Do not exceed the maximum safe working load.



# **HS295U ELECTRICAL SCOOTER**

### PRODUCT SPECIFICATIONS

Overall Length	102 cm or 40.2"
Overall Width (Wheel to Wheel)	64 cm or 24.6"
Overall Height	86 cm or 34"
Seat Depth	44.5 cm or 17.5"
Seat Width	43 cm or 17"
Seat Height	54 - 60 cm or 21.3 - 23.6"
Arm Rest Height (Floor to Arm Rest)	65 cm or 25.5"
Back Rest Height	44 cm or 17.3"
Front Wheel Diameter	22 cm or 8.5"
Back Wheel Diameter	22 cm or 8.5"
Safe Working Load	120 kg

Check our our full range of electrical wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

