


# KW-WF-LUX RECLINING GERIATRIC CHAIR



*For illustration purposes only*

## KW-WF-LUX RECLINING GERIATRIC CHAIR

### PRODUCT OVERVIEW

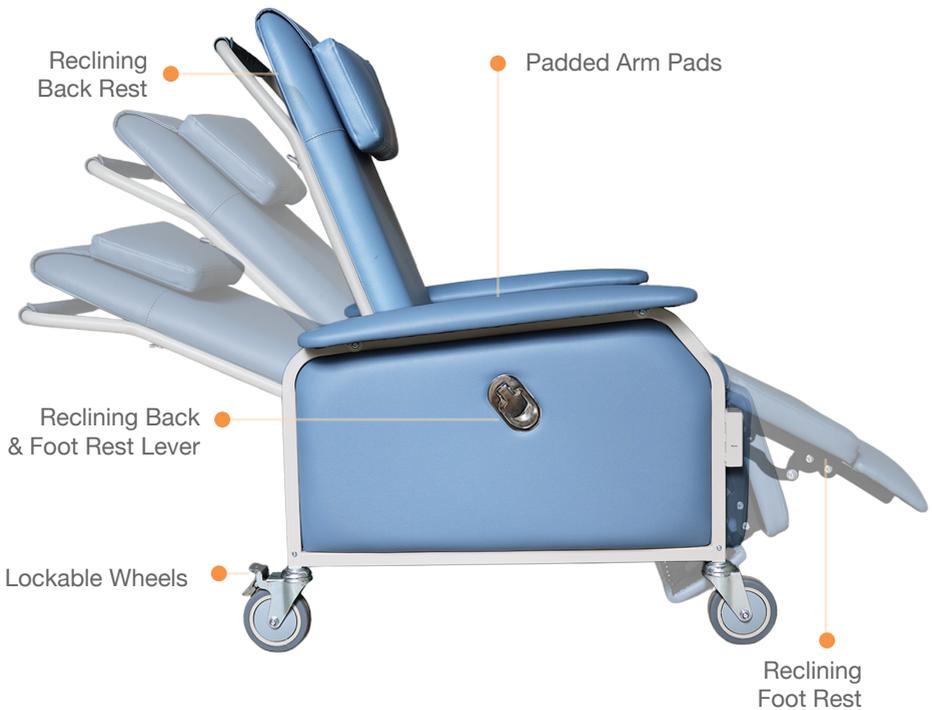

*For illustration purposes only*

Rainbow Care's Geriatric Chairs are equipped with large padded cushion for added comfort. It comes with a detachable table for convenience and multi-functionality. Built to recline from 90° to 160°, pull the knob to recline.

Designed according to ergonomic principles for users' comfort, this reclining geriatric chair is padded with cushion upholstery in its back rest, arm pads, seat, and foot rest. It can be conveniently adjusted to meet individuals' needs. Its versatility puts users and caretakers mind at ease.

## KW-WF-LUX RECLINING GERIATRIC CHAIR

### PRODUCT FEATURES



#### DETACHABLE TABLE

Pull the table outward to use. To close, press the raised button and fold the support bars down.



#### TABLE STORAGE

For storage, hang the table securely on the two support holders at the back of the chair.



#### RECLINING LEVER

Pull the lever to recline the chair's back & foot rest



Unlock



Lock

#### LOCKABLE WHEELS

Step on the top layer to unlock. Step on the bottom layer to lock.

*For illustration purposes only*

### CARE INSTRUCTIONS

- When using Reclining Geriatric Chair, please pay attention to people and/or items around to avoid collision
- Ensure the wheels are locked before getting in and out of the Geriatric Chair.
- Pay attention to the shift of the Center of Gravity when on elevated grounds
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the Geriatric Chair
- Do not exceed the maximum load

PRODUCT SPECIFICATIONS

**KW-WF-LUX RECLINING GERIATRIC CHAIR**

|                         |                                  |
|-------------------------|----------------------------------|
| Overall Length          | 96 cm or 37.7"                   |
| Overall Width           | 77.5 cm or 30.5"                 |
| Overall Height          | 120 cm or 47.2"                  |
| Seat Depth              | 46.5 cm or 18.3"                 |
| Seat Width              | 54 cm or 21.3"                   |
| Seat Height             | 55 cm or 21.7"                   |
| Wheel Size              | 12.7 cm or 5"                    |
| Table Overall Length    | 37.5 cm or 14.8"                 |
| Table Overall Width     | 65 cm or 25.6"                   |
| Table Overall Thickness | 4.1 cm or 1.6"                   |
| Materials               | Body: Steel<br>Seat: PVC Leather |
| Safe Working Load       | 200 kg                           |
| Overall Weight          | 44.4 kg                          |

Check out our full range of Reclining Geriatric Chairs. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

