

HOME FURNISHINGS | USER GUIDE

rainbow care

KW-WF RECLINING GERIATRIC CHAIRS



For illustration purposes only

KW-WF RECLINING GERIATRIC CHAIR

PRODUCT OVERVIEW



For illustration purposes only

Rainbow Care's Geriatric Chairs are equipped with large padded cushion for added comfort. It comes with a detachable table for convenience and multi-functionality. Built to recline from 90° to 130°, pull the knob to recline.

Designed according to ergonomic principles for users' comfort, this Reclining Geriatric Chair is padded with Cushion Upholstery in its Back Rest, Arm Pads, Seat, and Foot Rest. It can be conveniently adjusted to meet individuals' needs. Its Versatility puts users and caretakers mind at ease.

KW-WF RECLINING GERIATRIC CHAIR

PRODUCT FEATURES



HEIGHT ADJUSTMENT

Press the catch to adjust the height of the chair



RECLINING LEVER

Pull the lever to recline the chair's back & foot rest



LOCKABLE WHEELS

Push up to unlock and down to lock

For illustration purposes only

CARE INSTRUCTIONS

- When using Reclining Geriatric Chair, please pay attention to people and/or items around to avoid collision
- Ensure the wheels are locked before getting in and out of the Geriatric Chair.
- Ask for help if users have difficulty reclining the Geriatric Chair
- Use the safety belt to prevent user from sliding out of the chair
- Pay attention to the shift of the Center of Gravity when on elevated grounds
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the Geriatric Chair
- Do not exceed the maximum load

PRODUCT SPECIFICATIONS

KW-WF RECLINING GERIATRIC CHAIR

Overall Length	85 cm or 33"
Overall Width	69.5 cm or 27"
Overall Height	120 cm -126 cm or 47" - 50"
Seat Depth	54 cm or 21"
Seat Width	48 cm or 19"
Seat Height	45 cm - 51 cm or 18" - 20"
Table Overall Length	65 cm or 25.5"
Table Overall Width	34 cm or 13"
Table Overall Thickness	1.5 cm or 0.6"
Materials	Body: Steel Seat: PVC Leather
Safe Working Load	100 kg
Overall Weight	20 kg

Check out our full range of Reclining Geriatric Chairs. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

