

DETACHABLE COMMODE WITH SEAT BELT



DETACHABLE COMMODE WITH SEAT BELT

PRODUCT OVERVIEW



For illustration purposes only

Detachable Commode with Seat Belt

The Mobile Commode with Seat Belt is a versatile 4 in 1 care chair designed for shower use, bedside toileting, patient transport, and over toilet positioning. Built with a sturdy frame and smooth rolling castors, it allows safe and convenient mobility within home or clinical settings.



For illustration purposes only

DETACHABLE COMMODE WITH SEAT BELT

PRODUCT FEATURES



FOOT REST

Pull it out to use the foot rest. Make sure the front tube falls in the recess of the footrest support



ARM REST

Turn it up to make room for passage. Turn it down to sit comfortably. Press button and remove from side to detach.



LOCKABLE WHEELS

Push up to unlock and down to lock



REMOVABLE BACK REST

Push down buttons to unlock then pull upwards to detach.

For illustration purposes only

CARE INSTRUCTIONS

- When using mobility commode, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the commode.
- When getting in and out of the commode, please do not stand on the footplates.
- The footplates should be folded up beforehand, or removed.
- Pay attention to the shift of the center of gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the commode.

PRODUCT SPECIFICATIONS

Overall Length	64 - 91 cm or 25.2" - 35.8"
Overall Width	59.8cm or 23.5"
Overall Height	106 cm or 41.7"
Seat Depth	43 cm or 17"
Seat Width	48 cm or 19"
Seat Height	54.5 cm or 21.5"
Back Rest Height	47 cm or 18.5"
Arm Rest Height	76.9 cm or 30.3"
Front Wheel Diameter	12.7 cm or 5"
Back Wheel Diameter	12.7 cm or 5"
Safe Working Load	136 kg
Overall Weight	8.8 kg

Check our our full range of mobile commodes. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

