

BATHROOM AIDS | USER GUIDE

rainbow care

DETACHABLE MOBILE COMMUNE



For illustration purposes only

DETACHABLE MOBILE COMMODE

PRODUCT OVERVIEW

FS692

KY692L



As a 3-in-1 Detachable Mobile Commode, a brilliant multi-tasker: a Commode Chair, a Shower Chair, as well as an Indoor Wheelchair.

- Foam Seat Cushions
- Lightweight Frame
- Detachable Bucket
- Detachable Foot Rest
- Flip Up Arm Rest
- 4 Wheels Lock

DETACHABLE MOBILE COMMODE

PRODUCT FEATURES



LEG REST

Pull the catch release to remove detachable leg rest



ARM REST

Press button to release lock and flip up arm rest



LOCKABLE WHEELS

Push up to unlock and down to lock

For illustration purposes only

CARE INSTRUCTIONS

- When using mobility commode, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Commode.
- When getting in and out of the commode, please do not stand on the footplates.
- The footplates should be folded up beforehand, or removed.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the Commode.
- Do not exceed the maximum load.

PRODUCT SPECIFICATIONS	FS692 CHROME	KY692L ALUMINIUM
Overall Length	88 cm or 34.6"	86 cm or 33.9"
Overall Width	54 cm or 21.3"	55 cm or 21.7"
Overall Height	95 cm or 37.4"	96 cm or 37.8"
Seat Depth	43 cm or 17"	43 cm or 17"
Seat Width	43 cm or 17"	43 cm or 17"
Seat Height (Without Cushion)	46 cm or 18.1"	47 cm or 18.5"
Seat Height (With Cushion)	53 cm or 20.9"	53 cm or 20.9"
Front Wheel Diameter	12.5 cm or 5"	12.5 cm or 5"
Back Wheel Diameter	12.5 cm or 5"	12.5 cm or 5"
Safe Working Load	100 kg	90 kg
Overall Weight	12 kg	10 kg

Check our our full range of mobile commodes. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

