

FS696L ALUMINIUM MOBILE COMMODE



MOBILE COMMODE

PRODUCT OVERVIEW



For illustration purposes only



For illustration purposes only

FS696L Aluminium

It is made of lightweight aluminium frame and functions as a 2 in 1 commode where it is used as a commode chair and a shower chair.

- Foldable
- Detachable Bucket
- Height Adjustable

MOBILE COMMODE

PRODUCT FEATURES



HEIGHT ADJUSTMENT

Press the button and pull / push to the level that you want



REMOVE BUCKET

Lift up the seat cover to remove the bucket



LOCKABLE WHEELS

Press down to lock and lift up to unlock

For illustration purposes only

CARE INSTRUCTIONS

- When using mobility commode, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Commode.
- When getting in and out of the commode, please do not stand on the footplates.
- The footplates should be folded up beforehand, or removed.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the Commode.
- Do not exceed the maximum load.

PRODUCT SPECIFICATIONS

FS696L ALUMINIUM

Overall Length

61 cm or 24"

Overall Width

55 cm or 21.7"

Overall Height

81 cm - 91 cm
31.9" - 35.8"

Seat Depth

40 cm or 15.7"

Seat Width

38 cm or 15.0"

Seat Height

47.5 cm - 57.5 cm
18.7" - 22.6"

Front Wheel Diameter

10.2 cm or 4"

Back Wheel Diameter

10.2 cm or 4"

Safe Working Load

75 kg

Overall Weight

6 kg

Check our our full range of mobile commodes. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

