

LW06204C

MOBILE TRANSFER AID



LW06204C MOBILE TRANSFER AID

PRODUCT OVERVIEW



LW06204C Mobile Transfer Aid

- Easy to Use Patient Transfer
- Steel Frame
- Flip Up Foot Rest
- Detachable Bucket
- Safety Belt Included
- 4 Wheels Lock
- Supports up to 110 kg
- Adjustable Height

LW06204C MOBILE TRANSFER AID

PRODUCT FEATURES





SAFETY BUCKLE

Secure the bucket for an added layer of safety.



HYDRAULIC HEIGHT ADJUSTMENT

Step on the pedal to raise the seat height, or press and hold to lower it.

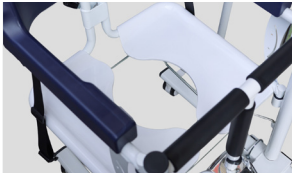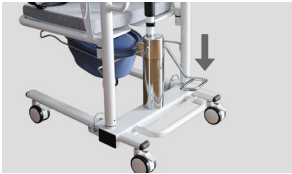

REMOVABLE COMMODE BUCKET

Insert or remove the commode bucket as needed.



LOCKABLE WHEELS

Step on the brake to lock the wheels and release to unlock.



CARE INSTRUCTIONS

- Be vigilant of your surroundings to prevent any collisions while using the patient transfer aid.
- Ensure the wheels are locked before getting in and out the patient transfer aid.
- Routinely clean the transfer aid to eliminate dirt, dust, and possible contaminants.
- Follow the weight capacity limits specified by the manufacturer
- Adhere to the specified weight capacity to avoid damage or danger
- Regularly adjust for user comfort—seat cushion and height, etc.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.

For illustration purposes only

LW06204C MOBILE TRANSFER AID

PRODUCT SPECIFICATIONS

Overall Length	76.5 cm / 30.1"
Overall Width	54 cm / 21.3"
Overall Height	92.6 - 108.6 cm / 36.5" - 42.8"
Seat Depth	39.5 cm / 15.6"
Seat Width	45 cm / 17.7"
Seat Height	43 - 59 cm / 16.9" - 23.2" (Without Cushion) 48 - 64 cm / 18.9" - 25.2" (With Cushion)
Front Wheel Diameter	7.5 cm / 3"
Back Wheel Diameter	7.5 cm / 3"
Frame Material	Steel
Cushion Cover Material	Foam
Overall Weight	32 kg
Safe Working Load	Up to 110 kg
Country Of Origin	Made in China

Check our our full range of patient hoist. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

