


NORMANDIE RECLINING GERIATRIC CHAIRS



For illustration purposes only

NORMANDIE GERIATRIC CHAIR

PRODUCT OVERVIEW

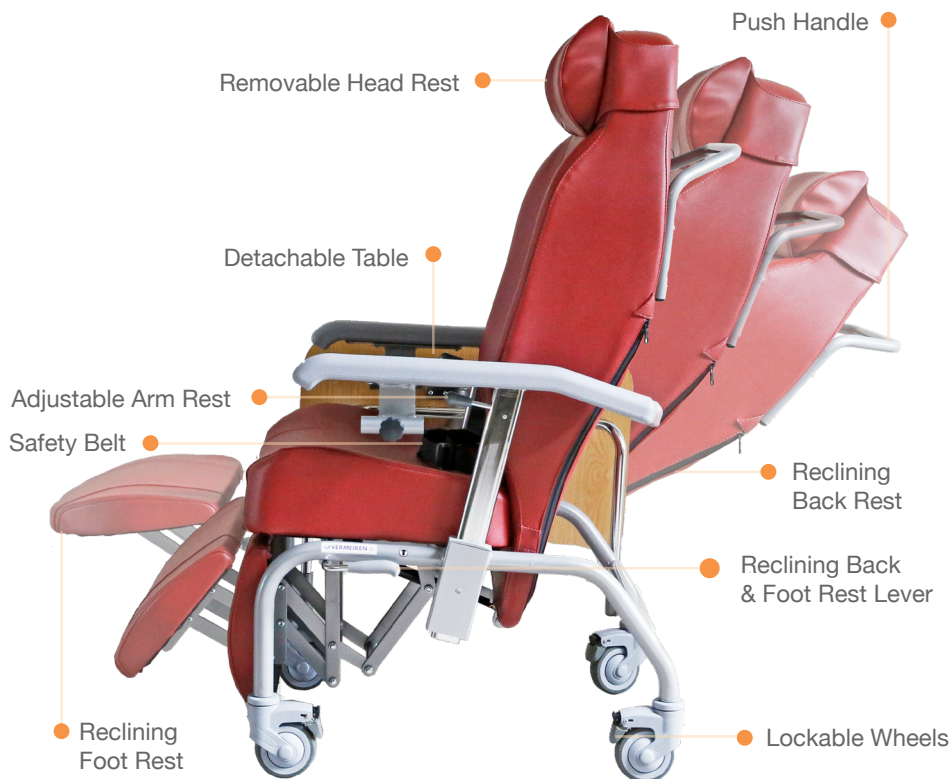

For illustration purposes only

Normandie Geriatric Chair is a Modern Take on a Classic, designed for the user's needs and comfort. Perfect in its contours, it features a maximum backrest recline of 130° with accompanying extended Foot Rest, Detachable Table, comfortable high-density foam upholstery and generous seating space.

It comes with Seat Belt and Wheel Locks on all four castors to ensure it stays in a secure position. Arm Rests are height adjustable to suit unique individuals' preference. Head Rest can also be easily removed and adjusted accordingly.

NORMANDIE GERIATRIC CHAIR

PRODUCT FEATURES




RECLINING LEVER

Pull the lever to recline the chair's back & foot rest



ADJUSTABLE ARM REST

Pull the lever to adjust the height of the arm rest

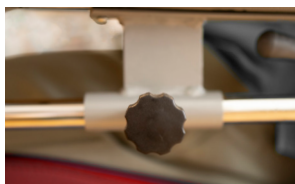

DETACHABLE TABLE

Insert table into the two holes underneath the arm rest then tighten the two knobs to secure it



LOCKABLE WHEELS

Push up to unlock and down to lock

For illustration purposes only

CARE INSTRUCTIONS

- Ensure the wheels are locked before getting in and out of the Geriatric Chair.
- Use the safety belt to prevent user from sliding out of the chair
- Pay attention to the shift of the Center of Gravity when on elevated grounds
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the Geriatric Chair
- Do not exceed the maximum load

PRODUCT SPECIFICATIONS

NORMANDIE GERIATRIC CHAIR

Overall Length	78 cm or 30.7"
Overall Width	82.5 cm or 32.5" (w/ Table) / 72 cm or 28.3" (w/o Table)
Overall Height	123.5 cm or 48.6"
Seat Depth	44 cm or 17"
Seat Width	54 cm or 21"
Seat Height	53 cm or 20.9"
Table Overall Length	40 cm or 15.7"
Table Overall Width	64.5 cm or 25.4"
Table Overall Thickness	1.6 cm or 0.6"
Materials	Body: Steel Seat: PVC Leather
Safe Working Load	130 kg
Overall Weight	25 kg

Check out our full range of Reclining Geriatric Chairs. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

