

NORMANDIE RECLINING GERIATRIC CHAIRS



NORMANDIE GERIATRIC CHAIR

PRODUCT OVERVIEW



For illustration purposes only

Normandie Geriatric Chair is a modern take on a classic, designed with the user's needs and comfort in mind. Its contoured design features a backrest that reclines up to 130°, an extendable footrest, detachable table, comfortable high-density foam upholstery and generous seating space.

It comes with a 3-point seat belt and wheel locks on all four castors to help keep the chair securely in position. The armrests are height adjustable to suit individual preferences, while the headrest can be easily removed and adjusted as needed.


NORMANDIE GERIATRIC CHAIR

PRODUCT FEATURES




RECLINING LEVER

Pull the lever to recline the chair's back & foot rest

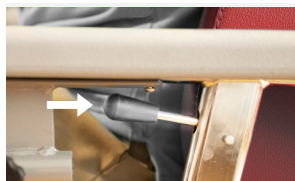

ADJUSTABLE ARM REST

Pull the lever to adjust the height of the arm rest

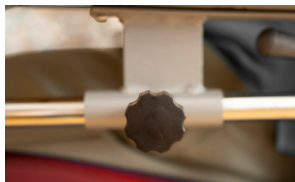

DETACHABLE TABLE

Insert table into the two holes underneath the arm rest then tighten the two knobs to secure it



BELT LENGTH LOCK

Slide the buckle to lock the belt at the adjusted length and prevent it from retracting.

For illustration purposes only

CARE INSTRUCTIONS

- Ensure the wheels are locked before getting in and out of the Geriatric Chair.
- Use the safety belt to prevent user from sliding out of the chair
- Pay attention to the shift of the Center of Gravity when on elevated grounds
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the Geriatric Chair
- Do not exceed the maximum load

PRODUCT SPECIFICATIONS

NORMANDIE GERIATRIC CHAIR

Overall Length	83 - 147 cm or 32.7 - 57.9"
Overall Width	68.5 cm or 27"
Overall Height	118 - 123 cm or 46.5 - 48.4"
Seat Depth	44 cm or 17.3"
Seat Width	54 cm or 21.3"
Seat Height	48.8 / 51.3 / 53.8 cm or 19.2 / 20.2 / 21.2"
Table Overall Length	40 cm or 15.7"
Table Overall Width	65.5 cm or 25.8"
Table Overall Thickness	1.6 cm or 0.6"
Materials	Body: Steel Seat: PVC Leather
Safe Working Load	130 kg
Overall Weight	25 kg

Check out our full range of Reclining Geriatric Chairs. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

