


QDC-501B ALTERNATING PRESSURE MATTRESS

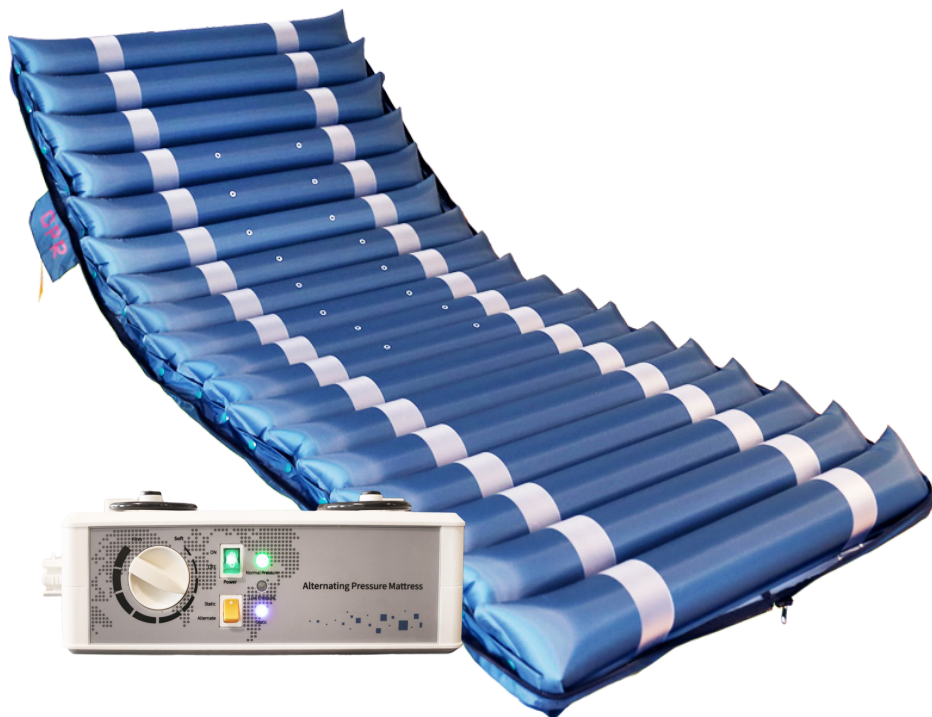

QDC- 501B ALTERNATING PRESSURE MATTRESS

PRODUCT OVERVIEW



For illustration purposes only

2:1 Inflate & Deflate Cell Structure Alternating Every 15 Minutes

- Mode: Alternate (2Hard:1Soft) & Static Pressure
- Top 3 tubes remains firm, remaining tube alternates throughout mattress

PRODUCT FEATURES



Emergency Safety Latch for CPR

CPR Function
deflates Mattress
during Emergencies



Nylon PVC Cover with Zipper

Water Resistant
Cover for easy
cleaning

QDC- 501B ALTERNATING PRESSURE MATTRESS

PRODUCT FEATURES



ON / OFF SWITCH

FRONT

PRESSURE DIAL

Adjustable Pressure Setting that adjusts mattress's soft tubes only. Alternates every 15 minutes.



MODE OPTIONS

Alternate & Static Mode Options To Choose From

AIR PRESSURE VENT

SIDE

Connect the air hoses from the mattress to the air pressure vent



HOOKS TO HANG

TOP

Hang the pump at the end of the bed or place it on a flat surface

CARE INSTRUCTIONS

- Keep the pump away from heated surfaces
- Ask for help if users have difficulty adjusting the Pressure Relief Mattress
- Ensure the Pressure Relief Mattress is resting on an even surface
- Clean the mattress while it's inflated with anti-bacterial wet wipes
- Avoid brushes with coarse bristles or caustic chemicals like bleach and ammonia
- Bouncing or flopping down on the mattress may also stresses the seams
- Do not exceed the maximum load

PRODUCT SPECIFICATIONS

QDC- 501B ALTERNATING PRESSURE MATTRESS

PRESSURE MATTRESS

| | |
|-------------------|-----------------|
| Overall Length | 200 cm or 78.7" |
| Overall Width | 90 cm or 35.4" |
| Overall Height | 11.5 cm or 4.5" |
| Pressure Range | 50 - 120 mmHg |
| Safe Working Load | 40 - 120 kg |
| Overall Weight | 5.7 kg |

PUMP

| | |
|-------------------------|---------------------------|
| Overall Length | 31 cm or 12.2" |
| Overall Width | 11.5 cm or 4.5" |
| Overall Height | 14 cm or 5.5" |
| Electrical Requirements | Electrical AV 110 / 220 V |

Check out our full range of Hospital Beds & Mattresses. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

