

WALKING AIDS | USER GUIDE

rainbow care

FS965LH-F6 ROLLATOR



For illustration purposes only

FS965LH-F6 ROLLATOR

PRODUCT OVERVIEW



PRODUCT OVERVIEW



FS965LH-F6 Rollator

It is a walking aid meant to assist users to walk independently and with stability.

With the flip up Seat Cushion, it allows users to take a rest whenever.

It also comes with a basket storage for users' convenience.

FS965LH-F6 ROLLATOR

PRODUCT FEATURES




HANDLE BRAKES

Squeeze to brake, push down to lock.



ADJUSTABLE HEIGHT

Flip lever up to unlock, adjust height and flip down to lock in place.

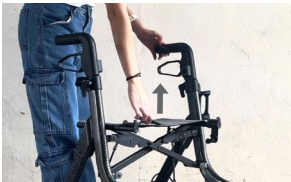

FOLDABLE

Pull the center strap upward to fold the rollator.

For illustration purposes only

CARE INSTRUCTIONS

- When using rollator, please pay attention to people and/or items around to avoid collision.
- Make sure the level of the height of each handle is the same before using it.
- Make sure the level of flooring is even before seating down.
- Do not exceed the maximum load.

FS965LH-F6 ROLLATOR

PRODUCT SPECIFICATIONS

Overall Length	64 cm / 25.2"
Overall Width	60.2 cm / 23.7
Overall Height	81 - 93 cm / 31.9 - 36.6"
Folded Length	64 cm / 25.2"
Folded Width	25 cm / 9.8"
Folded Height	81 cm / 31.9"
Seat Depth	24.5 cm / 9.7"
Seat Width	42.5 cm / 16.7" (Cushion)
Seat Height	52 cm / 20.5" (Floor to Seat)
Front Wheel Diameter	20.5 cm / 8"
Back Wheel Diameter	17.5 cm / 6.9"
Safe Working Load	80 kg 5 kg (Basket)
Overall Weight	9 kg

Check our our full range of rollators. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

