



# PRODUCT OVERVIEW



## PRODUCT OVERVIEW



#### FS969 3 Wheel Rollator

It is a walking aid meant to assist users to walk independently and with stability.

It also comes with a basket storage for users' convenience.

For illustration purposes only



### **PRODUCT FEATURES**





# LOCKABLE HANDLE BRAKES

Push down to lock the rollator in place.





### **REMOVABLE BASKET**

Lift up to remove basket.





### **FOLDABLE**

Pull the center knob towards you to fold.

For illustration purposes only

### **CARE INSTRUCTIONS**

- When using rollator, please pay attention to people and/or items around to avoid collision.
- Make sure the level of the height of each handle is the same before using it.
- · Make sure the level of flooring is even before seating down.
- · Do not exceed the maximum load.

### PRODUCT SPECIFICATIONS

Overall Length	58.5 cm / 23"
Overall Width	60.5 cm / 23.8"
Overall Height	83 - 98 cm / 32.7 - 38.6"
Folded Length	58.5 cm / 23"
Folded Width	24.5 cm / 9.6"
Folded Height	83 cm / 32.7"
Front Wheel Diameter	20 cm / 7.9"
Back Wheel Diameter	20 cm / 7.9"
Safe Working Load	80 kg 5kg (Basket)
Overall Weight	6 kg
Country Of Origin	Made in China

Check our our full range of rollators. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.



