

WALKING AIDS | USER GUIDE

rainbow care

FOUR-LIGHT ROLLATOR



For illustration purposes only

FOUR-LIGHT 4 WHEEL ROLLATOR

PRODUCT OVERVIEW



PRODUCT OVERVIEW



Four-Light 4 Wheel Rollator

It is a walking aid meant to assist users to walk independently and with stability.

With the flip up Seat Cushion, it allows users to take a rest whenever.

It also comes with a basket storage for users' convenience.

FOUR-LIGHT 4 WHEEL ROLLATOR

PRODUCT FEATURES



HEIGHT ADJUSTMENT

Press and hold the red button and adjust height.



REMOVABLE WHEELS

Press red button to release wheels and remove.



REMOVABLE SEAT SUPPORT

Unscrew knob to remove seat support.

For illustration purposes only

CARE INSTRUCTIONS

- When using rollator, please pay attention to people and/or items around to avoid collision.
- Make sure the level of the height of each handle is the same before using it.
- Make sure the level of flooring is even before seating down.
- Do not exceed the maximum load.

BYA3105 ROLLATOR

PRODUCT SPECIFICATIONS

Overall Length	83 cm / 32.7"
Overall Width	67 cm / 26.4"
Overall Height	89 - 99 cm / 35 - 39 "
Folded Length	73.5 cm / 28.9"
Folded Width	27 cm / 10.6"
Folded Height	89 cm / 35"
Seat Depth	25 cm / 9.8"
Seat Width	44.5 cm / 17.5"
Seat Height	54 cm / 21.3"
Front Wheel Diameter	20 cm / 7.9"
Back Wheel Diameter	17.5 cm / 6.9"
Safe Working Load	130 kg 5kg (Basket)
Overall Weight	8 kg

Check our our full range of rollators. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

