

FS965LH-1 ROLLATOR



FS9125LH-1 ROLLATOR

PRODUCT OVERVIEW



PRODUCT OVERVIEW



FS9125LH-1 Rollator

It is a walking aid meant to assist users to walk independently and with stability.

With the flip up Seat Cushion, it allows users to take a rest whenever.

It also comes with a basket storage for user's convenience.


FS9125LH-1 ROLLATOR

PRODUCT FEATURES



REMOVE THE BASKET

Lift up the seat and pull up the basket

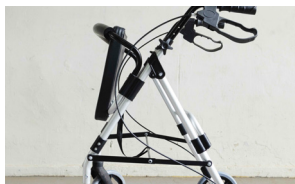

FOLDABLE

Remove the basket and pull the string in the middle upward to fold

For illustration purposes only

CARE INSTRUCTIONS

- When using rollator, please pay attention to people and/or items around to avoid collision.
- Make sure the level of the height of each handle is the same before using it.
- Make sure the level of flooring is even before seating down.
- Do not exceed the maximum load.

FS9125LH-1 ROLLATOR

PRODUCT SPECIFICATIONS

Overall Length	67 cm or 26.4"
Overall Width	56 cm or 22"
Overall Height	52 cm or 20.5"
Folded Length	56 cm or 22"
Folded Width	10 cm or 3.9"
Folded Height	88 cm or 34.6"
Seat Depth	33 cm or 13"
Seat Width	36 cm or 14" (Cushion)
Seat Height	54 cm or 21.3" (Floor to Seat) 35 cm or 13.8" (Floor to Basket Base)
Front Wheel Diameter	20 cm or 7.9"
Back Wheel Diameter	20 cm or 7.9"
Safe Working Load	80 kg 5 kg (Basket)
Overall Weight	7.4 kg

Check our our full range of rollators. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

