

WALKING AIDS | USER GUIDE

rainbow care

FS965LH-B ROLLATOR



For illustration purposes only

FS965LH-B ROLLATOR

PRODUCT OVERVIEW



PRODUCT OVERVIEW



FS965LH-B Rollator

It is a walking aid meant to assist users to walk independently and with stability.

With the flip up Seat Cushion and Foot Rest, allow users to take a rest whenever. It also comes with a basket storage for user's convenience.


FS965LH-B ROLLATOR

PRODUCT FEATURES




REMOVE THE BASKET

Lift up the seat and pull up the basket

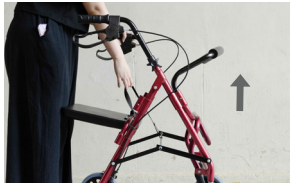

FOLDABLE

Remove the basket and pull the string in the middle upward to fold

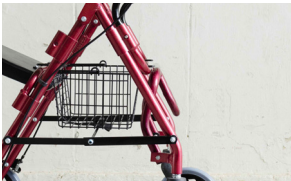

FLIP UP FOOT REST

Flip up to keep or flip down for usage



CHANGEABLE SEAT SUPPORT

Press the button and pull it up to change the direction

For illustration purposes only

CARE INSTRUCTIONS

- When using rollator, please pay attention to people and/or items around to avoid collision.
- Make sure the level of the height of each handle is the same before using it.
- Make sure the level of flooring is even before seating down.
- Do not exceed the maximum load.

FS965LH-B ROLLATOR

PRODUCT SPECIFICATIONS

Overall Length	78.5 cm / 30.9"
Overall Width	58.5 cm / 23"
Overall Height	77 - 88 cm / 30.3 - 34.6"
Folded Length	34.5 cm / 13.7"
Folded Width	58.5 cm / 23"
Folded Height	90.5 cm / 35.6"
Seat Depth	31.5 cm / 12.4"
Seat Width	34.5 cm / 13.6" (Cushion)
Seat Height	53.5 cm / 21" (Floor to Seat)
Front Wheel Diameter	20 cm / 7.9"
Back Wheel Diameter	20 cm / 7.9"
Safe Working Load	80 kg 5 kg (Basket)
Overall Weight	8.8 kg

Check our our full range of rollators. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

