

WALKING AIDS | USER GUIDE

rainbow care

FS965LH-B ROLLATOR



For illustration purposes only

FS965LH-B ROLLATOR

PRODUCT OVERVIEW



PRODUCT OVERVIEW



FS965LH-B Rollator

It is a walking aid meant to assist users to walk independently and with stability.

With the flip up Seat Cushion and Foot Rest, allow users to take a rest whenever. It also comes with a basket storage for user's convenience.


FS965LH-B ROLLATOR

PRODUCT FEATURES




REMOVE THE BASKET

Lift up the seat and pull up the basket

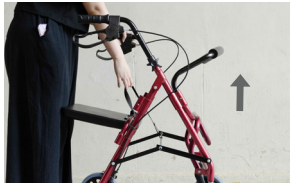

FOLDABLE

Remove the basket and pull the string in the middle upward to fold

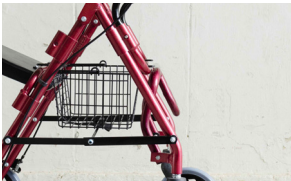

FLIP UP FOOT REST

Flip up to keep or flip down for usage



CHANGEABLE SEAT SUPPORT

Press the button and pull it up to change the direction

For illustration purposes only

CARE INSTRUCTIONS

- When using rollator, please pay attention to people and/or items around to avoid collision.
- Make sure the level of the height of each handle is the same before using it.
- Make sure the level of flooring is even before seating down.
- Do not exceed the maximum load.

FS965LH-B ROLLATOR

PRODUCT SPECIFICATIONS

Overall Length	77 cm or 30.3"
Overall Width	58 cm or 22.8"
Overall Height	88 cm or 34.6"
Folded Length	44 cm or 17.3"
Folded Width	58 cm or 22.8"
Folded Height	104 cm or 40.9"
Seat Depth	31 cm or 12.2"
Seat Width	35 cm or 13.8" (Cushion)
Seat Height	53 cm or 20.9" (Floor to Seat) 35 cm or 13.8" (Floor to Basket Base)
Front Wheel Diameter	20 cm or 7.9"
Back Wheel Diameter	20 cm or 7.9"
Safe Working Load	80 kg 5 kg (Basket)
Overall Weight	8.8 kg

Check our our full range of rollators. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

