

WALKING AIDS | USER GUIDE

rainbow care

FS966LH-B ROLLATOR



For illustration purposes only

FS966LH-B ROLLATOR

PRODUCT OVERVIEW



PRODUCT OVERVIEW



FS966LH-B Rollator

It is a walking aid meant to assist users to walk independently and with stability.

With the flip up Seat Cushion and Foot Rest, allow users to take a rest whenever. It also comes with a basket storage for user's convenience.

FS966LH-B ROLLATOR

PRODUCT FEATURES



REMOVE THE BASKET

Lift up the seat and pull up the basket




FOLDABLE

Remove the basket and pull the string in the middle upward to fold



HEIGHT ADJUSTMENT

Turn the knob, remove the screw to change the backrest

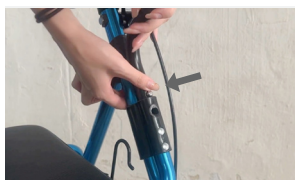

CHANGEABLE SEAT SUPPORT

Press the button and pull it up to change the direction



For illustration purposes only

CARE INSTRUCTIONS

- When using rollator, please pay attention to people and/or items around to avoid collision.
- Make sure the level of the height of each handle is the same before using it.
- Make sure the level of flooring is even before seating down.
- Do not exceed the maximum load.

FS965LH-B ROLLATOR

PRODUCT SPECIFICATIONS

Overall Length	63 cm / 24.8"
Overall Width	60 cm / 23.6"
Overall Height	79.5 - 89 cm / 31.3 - 35"
Folded Length	49 cm / 12.3"
Folded Width	60 cm / 23.6"
Folded Height	91 cm / 35.8"
Seat Depth	32.3 cm / 12.7"
Seat Width	35.5 cm / 14" (Cushion)
Seat Height	50.5 cm / 19" (Floor to Seat)
Front Wheel Diameter	15 cm / 6"
Back Wheel Diameter	15 cm / 6"
Safe Working Load	80 kg 5 kg (Basket)
Overall Weight	8.5 kg

Check our our full range of rollators. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

