

WALKING AIDS | USER GUIDE

rainbow care

QUAVA 4 WHEEL ROLLATOR



For illustration purposes only

QUAVA 4 WHEEL ROLLATOR

PRODUCT OVERVIEW



PRODUCT OVERVIEW



Quava 4 Wheel Rollator

It is a walking aid meant to assist users to walk independently and with stability.

With the flip up Seat Cushion, it allows users to take a rest whenever.

It also comes with a basket storage for users' convenience.

QUAVA 4 WHEEL ROLLATOR

PRODUCT FEATURES



FOLDABLE

Pull the center strap upward to fold the rollator.



LOCKABLE HANDLE BRAKES

Push down to lock the rollator in place.



REMOVABLE SEAT SUPPORT

Squeeze and pull up to remove seat support.



For illustration purposes only

CARE INSTRUCTIONS

- When using rollator, please pay attention to people and/or items around to avoid collision.
- Make sure the level of the height of each handle is the same before using it.
- Make sure the level of flooring is even before seating down.
- Do not exceed the maximum load.

QUAVA 4 WHEEL ROLLATOR

PRODUCT SPECIFICATIONS

Overall Length	79 cm / 31.1"
Overall Width	65 cm / 25.6"
Overall Height	83 - 96 cm / 32.7 - 37.8"
Folded Length	90 cm / 35.4"
Folded Width	25.5 cm / 10"
Folded Height	34 cm / 13.4"
Seat Depth	21.8 cm / 8.6"
Seat Width	45 cm / 17.7"
Seat Height	53.5 cm / 21.1"
Front Wheel Diameter	20 cm / 7.9"
Back Wheel Diameter	20 cm / 7.9"
Safe Working Load	136 kg 5 kg (Basket)
Overall Weight	7 kg

Check our our full range of rollators. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

