

ST003A STAIRS CLIMBING WHEELCHAIR



ST003A STAIRS CLIMBING WHEELCHAIR

PRODUCT OVERVIEW



ST003A Stairs Climbing Wheelchair

The ST003A Stairs Climbing Wheelchair is a fully electrical wheelchair with a unique track structure. It allows for safe and convenient transfers, and can be operated by a single person.

Belt Mount and 4-Point Safety Buckle Seat Belt sold separately


ST003A STAIRS CLIMBING WHEELCHAIR

PRODUCT FEATURES



TO POWER ON

Plug battery to control box, Turn battery key fully clockwise, Press red button (green light means on).

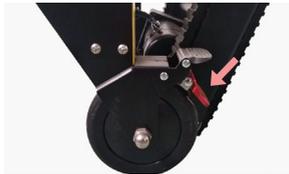

WHEEL BRAKES

Step down on rear wheel brakes to lock wheelchair in place.



OPEN TRACKS


Push down red handle bar to release tracks. Let go to lock in place.



FOLDING

Push up red handle bar, fold up seat and buckle in place to secure folding position.

REMOTE CONTROL GUIDE

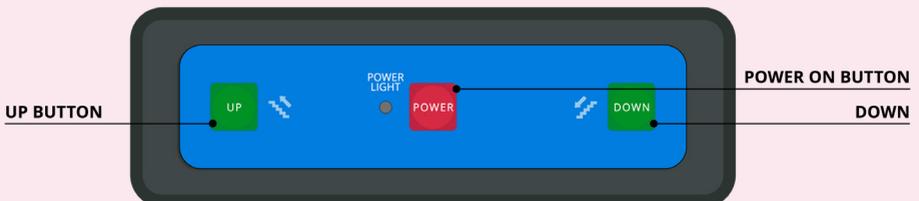

ST003A STAIRS CLIMBING WHEELCHAIR

PRODUCT SPECIFICATIONS

Overall Length	93.5 cm / 36.8 " (Lowest) 120.5 cm / 47.4 " (Highest)
Overall Width	49 cm / 19.3 "
Overall Height	124 cm / 48.8 " (Lowest) 158 cm / 62.2 " (Highest)
Folded Length	40 cm / 15.7 "
Folded Width	49 cm / 19.3 "
Folded Height	126 cm / 49.6 "
Safe Working Load	Up to 200 kg
Overall Weight	27 kg

CARE INSTRUCTIONS

- Check surroundings before moving to avoid collision.
- Secure the user with the safety belt before operation.
- Inspect tracks, wheels and controls before each use.
- Do not exceed the maximum safe working load.
- Operate slowly and steadily when climbing or descending stairs.
- Ensure sufficient space when passing through doors and landings.
- Keep hands, clothing and loose items away from moving parts.
- Ensure the battery is adequately charged before use. Avoid full discharge.
- Use the emergency stop or manual lowering only during power failure or emergencies.
- The stairs climbing function should be operated by a trained caregiver only.

Check our our full range of transfer aids. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

