

ST004 STAIRS EVACUATION CHAIR



For illustration purposes only

ST004 STAIRS EVACUATION WHEELCHAIR

PRODUCT OVERVIEW



ST004 Stairs Evacuation Chair

The ST004 Stairs Evacuation Chair was developed to evacuate people with reduced mobility. In an emergency, the rails create friction, act as a brake to slow down the descent, and allow smooth and controlled operation to ensure that customers are transported safely to a safe place .

ST004 STAIRS EVACUATION WHEELCHAIR

PRODUCT FEATURES




RELEASE TRACKS

Unclip the safety belt to release the chair tracks.



EXTEND HANDLES

Pull the chair handle towards you.

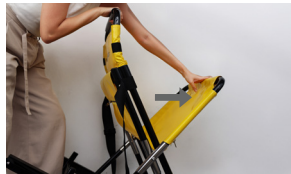

OPEN SEAT

Push the seat frame forward until fully extended.



LOCK FRAME

Raise the seat frame until it connects with the track strut.

CARE INSTRUCTIONS

- Inspect straps, handles, and tracks for damage before each use.
- Lock wheels during patient transfer to prevent unintended movement.
- Ensure the chair is fully unfolded and stable before use.
- Clean and disinfect after every use.
- Do not exceed the maximum safe working load.
- Familiarise operators with emergency features and proper usage in advance.

ST004 STAIRS EVACUATION CHAIR

PRODUCT SPECIFICATIONS

Overall Length	85 cm or 33.5"
Overall Width	52 cm or 20.5"
Overall Height	137 cm or 53.9"
Folded Length	110 cm or 43.3"
Folded Width	56 cm or 22.0"
Folded Height	18 cm / 7.1"
Seat Depth	48 cm or 18.8 "
Seat Width	43 cm or 16.9 "
Back Rest Height	51 cm / 20 " (Lowest) 96 cm / 37.7 " (Highest)
Seat Material	PVC
Frame Material	Aluminum Alloy
Overall Weight	10 kg
Safe Working Load	159 kg
Country Of Origin	Made in China

Check our our full range of stairs climbing wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

