

# STATIONARY COMMODE



## STATIONARY COMMODE

### 328L2 ALUMINIUM PRODUCT OVERVIEW



*For illustration purposes only*

### FT4213STEEL PRODUCT OVERVIEW



*For illustration purposes only*

# STATIONARY COMMODE

## 328L2 ALUMINIUM



## FT4213 STEEL



*For illustration purposes only*

### Stationary Commode

Made of lightweight aluminium frame and functions as a 2 in 1 commode where it is used as a commode chair and a shower chair. Foldable for storage.

## PRODUCT FEATURES



### HEIGHT ADJUSTMENT

Press the button and pull / push to the level that you want



### REMOVE BUCKET

328L2 - Lift up the seat cover to remove  
FT4213 - Slide out to remove

*For illustration purposes only*

PRODUCT SPECIFICATIONS	328L2	FT4213
Overall Length	56 cm or 22"	52 cm or 20.5"
Overall Width	57 cm or 22.4"	52.5 cm or 20.6"
Overall Height	87 cm - 91.5 cm 34.3" - 36"	68.5 cm - 78.5 cm 26.9" - 30.9"
Seat Depth	42 cm or 16.5"	42 cm or 16.5"
Seat Width	42 cm or 16.5"	42 cm or 16.5"
Seat Height	45 cm - 49.5 cm 17.7" - 19.5"	34.5 cm - 44.5 cm 13.6" - 17.5"
Safe Working Load	100 kg	80 kg
Overall Weight	5.5 kg	4.5 kg

## CARE INSTRUCTIONS

- When using stationary commode, please pay attention to people and/or items around to avoid collision.
- Make sure the level of the height of each leg is the same before sitting down.
- Do not exceed the maximum load.

Check our our full range of stationary commodes. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

