

# HEAVY DUTY STATIONARY COMMUNE



*For illustration purposes only*

## HEAVY DUTY STATIONARY COMMODE

### FT4203 STEEL PRODUCT OVERVIEW



*For illustration purposes only*

### FT4206 ALUMINIUM PRODUCT OVERVIEW



*For illustration purposes only*

## STATIONARY COMMODE

### FT4203 STEEL



*For illustration purposes only*

### Stationary Commode

### FT4206 ALUMINIUM



With functions as a 2 in 1 commode where it is used as a commode chair and a shower chair.

## PRODUCT FEATURES



### HEIGHT ADJUSTMENT

Press the button and pull / push to the level that you want



### REMOVE BUCKET

Lift up the seat cover to remove the bucket

*For illustration purposes only*

PRODUCT SPECIFICATIONS	FT4203	FT4206
Overall Length	50 cm or 19.7"	50 cm or 19.7"
Overall Width (Armrest - Armrest)	62 cm or 24.4"	67 cm or 26.4"
Overall Width (Leg - Leg)	72 cm or 28.3"	67 cm or 26.4"
Overall Height	64 cm - 74 cm 25.2" - 29.1"	77.5 cm - 87.5 cm 30.5" - 34.4"
Seat Depth	40 cm or 15.7"	41 cm or 16.9"
Seat Width (Internal)	20 cm or 7.8"	20 cm or 7.9"
Seat Width (External)	34 cm or 13.4"	34 cm or 13.4"
Seat Height	44 cm - 54 cm 17.3" - 21.3"	45 cm - 55 cm 17.7" - 21.7"
Safe Working Load	200 kg	200 kg
Overall Weight	9 kg	9 kg

## CARE INSTRUCTIONS

- When using stationary commode, please pay attention to people and/or items around to avoid collision.
- Make sure the level of the height of each leg is the same before sitting down.
- Do not exceed the maximum load.

Check our our full range of stationary commodes. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

