

FS894L

STATIONARY COMMODE



For illustration purposes only

STATIONARY COMMODE

FS894L ALUMINIUM PRODUCT OVERVIEW



For illustration purposes only



For illustration purposes only

Stationary Commode

Made of lightweight aluminium frame and functions as a 2 in 1 commode where it is used as a commode chair and a shower chair. Foldable for storage.

STATIONARY COMMODE

PRODUCT FEATURES



HEIGHT ADJUSTMENT

Press the button and pull / push to the level that you want



REMOVE BUCKET

Lift up the seat cover to remove the bucket

CARE INSTRUCTIONS

- Place the commode on a flat and stable surface before use.
- Ensure all legs are adjusted to the same height and securely locked before sitting.
- Check that all rubber tips are fitted securely and are not excessively worn.
- Ensure the seat, armrests and bucket are properly positioned and secure before use.
- Do not exceed the safe working load stated for the product.
- When transferring to or from the commode, ensure the chair remains stable and does not move.
- Empty and clean the commode bucket after use. Wash with mild soap and water and dry thoroughly before replacing.
- Clean the seat and frame regularly using a soft cloth and mild cleaning solution.
- Avoid abrasive cleaners that may damage the surface.
- Regularly inspect the frame, adjustment points, seat and other fittings for looseness, cracks, damage or wear.
- Stop using the commode if any part appears loose, damaged or unstable, and arrange for inspection or replacement before further use.

PRODUCT SPECIFICATIONS

FS894L

Overall Length	43 cm or 16.9"
Overall Width (Armrest - Armrest)	55 cm or 21.7"
Overall Width (Leg - Leg)	50 cm or 19.7"
Overall Height	76 cm - 86 cm 29.9" - 33.9"
Seat Depth	43 cm or 16.9"
Seat Width (Internal)	23 cm or 9.1"
Seat Width (External)	38 cm or 15"
Seat Height	52 cm - 62 cm 20.5" - 24.4"
Safe Working Load	80 kg
Overall Weight	5 kg

Check out our full range of stationary commodes. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task, such as, such as transferring someone, are performed.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

