

FT4206 HEAVY DUTY STATIONARY COMMODE



HEAVY DUTY STATIONARY COMMODE



For illustration purposes only

Stationary Commode

With functions as a 2 in 1 commode where it is used as a commode chair and a shower chair.



For illustration purposes only

HEAVY DUTY STATIONARY COMMODE

PRODUCT FEATURES



HEIGHT ADJUSTMENT

Press the button and pull / push to the level that you want



REMOVE BUCKET

Lift up the seat cover to remove the bucket

CARE INSTRUCTIONS

- Place the commode on a flat and stable surface before use.
- Ensure all legs are adjusted to the same height and securely locked before sitting.
- Check that all rubber tips are fitted securely and are not excessively worn.
- Ensure the seat, armrests and bucket are properly positioned and secure before use.
- Do not exceed the safe working load stated for the product.
- When transferring to or from the commode, ensure the chair remains stable and does not move.
- Empty and clean the commode bucket after use. Wash with mild soap and water and dry thoroughly before replacing.
- Clean the seat and frame regularly using a soft cloth and mild cleaning solution. Avoid abrasive cleaners that may damage the surface.
- Regularly inspect the frame, adjustment points, seat and other fittings for looseness, cracks, damage or wear.
- Stop using the commode if any part appears loose, damaged or unstable, and arrange for inspection or replacement before further use

PRODUCT SPECIFICATIONS

FT4206

Overall Length	50 cm or 19.7"
Overall Width (Armrest - Armrest)	67 cm or 26.4"
Overall Width (Leg - Leg)	67 cm or 26.4"
Overall Height	77.5 cm - 87.5 cm 30.5" - 34.4"
Seat Depth	41 cm or 16.1"
Seat Width (Internal)	20 cm or 7.9"
Seat Width (External)	34 cm or 13.4"
Seat Height	45 cm - 55 cm 17.7" - 21.7"
Safe Working Load	200 kg
Overall Weight	9 kg

Check out our full range of stationary commodes. When making comparisons, our best suggestion would be for you to think about your environment consider how certain tasks, such as transferring someone, are performed.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

