


ELATOR© ELECTRIC PUSH AID FOR WHEELCHAIRS AND ROLLATORS



ELATOR © ELECTRIC PUSH AID FOR WHEELCHAIRS AND ROLLATORS

PRODUCT OVERVIEW

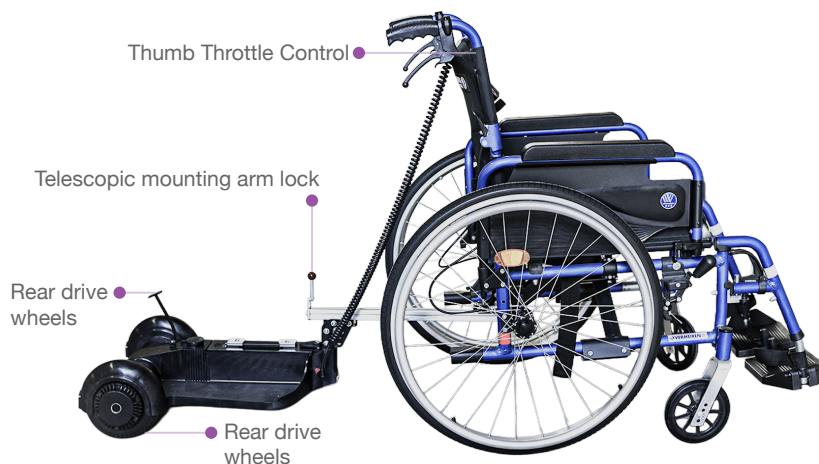

ELATOR © Electric Push Aid for Wheelchairs and Rollators

- Electric power assist for manual wheelchairs and rollators
- Reduces caregiver effort on slopes and longer distances
- Telescopic mounting arm with adjustable length
- Thumb throttle control for smooth speed adjustment
- Three selectable speed levels

For illustration purposes only


ELECTRIC PUSH AID FOR WHEELCHAIRS AND ROLLATORS

PRODUCT FEATURES



REMOVE PUSH AID

Press on blue button to release, pull upwards to unlock from equipment

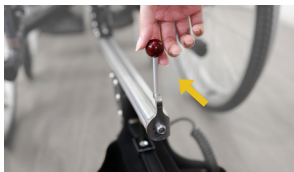

LOCK AND ATTACH

Rotate to side to unlock, rotate upwards to lock.



SPEED SETTINGS

Push down throttle to move. Press button once to adjust speed, three times consecutively to reverse.

For illustration purposes only

CARE INSTRUCTIONS

- Wipe clean with a soft dry or slightly damp cloth
- Do not use abrasive cleaners, solvents, or strong chemicals
- Avoid exposure to heavy rain or direct water spray
- Keep electrical components dry at all times
- Store in a cool and dry place when not in use
- Charge the battery according to the user manual instructions
- Ensure the unit is switched off when charging
- Remove debris from wheels regularly for smooth operation
- Check that the mounting attachment is secure before each use

ELATOR © ELECTRIC PUSH AID FOR WHEELCHAIRS AND ROLLATORS

PRODUCT SPECIFICATIONS

Overall Length	57 cm or 22.4"
Overall Width (Wheel to Wheel)	39 cm or 15.4"
Overall Height	34.5 cm or 13.6"
Speed	Maximum Speed: Up to 6 km per hour Speed Settings: 3 levels
Battery	Type: Rechargeable lithium battery Charging Method: External charger
Safe Working Load	Up to 130 kg
Overall Weight	Approx 8.5 kg including battery
Country Of Origin	Designed and Manufactured in Germany

Check our our full range of Wheelchair Accessories When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.



☎ 6223 0904 • 6931 7425 ☎ 8501 1878 ✉ sales@rainbowcare.com.sg 🌐 rainbowcare.com.sg
📘 @rainbowcare 📷 rainbowcaresg 📍 Blk 3007 Ubi Road 1 #02-410 Singapore 408701