

BOBBY 16 LIGHTWEIGHT WHEELCHAIR



For illustration purposes only

BOBBY 16 LIGHTWEIGHT WHEELCHAIR

PRODUCT OVERVIEW



Lightweight Wheelchair

- Easy to fold and operate
- Multi-function wheelchair with fold-up foot rests for easier patient transfer.
- Foldable back rest for easy storage
- Ventilated and cushioned seat upholstery
- Aluminium lightweight frame



For illustration purposes only

LIGHTWEIGHT WHEELCHAIR

PRODUCT FEATURES



DETACHABLE ARM REST

Pinch and lift the lever to release the armrest lock, then pull upward to detach



CAR BOOT FRIENDLY

Collapse wheelchair into a square to keep in your car boot



FLIP UP FOOT REST

Flip up foot rest by pushing upwards

For illustration purposes only

CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

BOBBY 16 LIGHTWEIGHT WHEELCHAIR

PRODUCT SPECIFICATIONS

| | |
|-------------------------------------|------------------|
| Overall Length | 92.5 cm or 36.4" |
| Overall Width (Wheel to Wheel) | 58.5 cm or 23" |
| Overall Height | 92 cm or 70" |
| Folded Length | 74.5 cm or 29" |
| Folded Width (Wheel to Wheel) | 34 cm or 13.4" |
| Folded Height | 72.5 cm or 28.5" |
| Seat Depth | 40 cm or 15.8" |
| Seat Width | 47 cm or 18.5" |
| Seat Height (Floor to Seat Base) | 50.5 cm or 19.9" |
| Arm Rest Height (Floor to Arm Rest) | 71 cm or 28" |
| Back Rest Height | 40 cm or 15.7" |
| Front Wheel Diameter | 15 cm or 6" |
| Back Wheel Diameter | 40 cm or 15.7" |
| Safe Working Load | 115 kg |
| Overall Weight | 13.5 kg |

Check our our full range of lightweight wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

