

WHEELCHAIRS | USER GUIDE

rainbow care

# BOBBY ASTRO 20 LIGHTWEIGHT WHEELCHAIR



*For illustration purposes only*

**VERMEIREN**

## BOBBY ASTRO 16 LIGHTWEIGHT WHEELCHAIR

### PRODUCT OVERVIEW



### Lightweight Wheelchair

- Easy to fold and operate
- Multi-function wheelchair with detachable foot rests for easier patient transfer.
- Foldable back rest for easy storage
- Ventilated and cushioned seat upholstery
- Aircraft grade aluminium lightweight frame
- Designed in Belgium

## VERMEIREN LIGHTWEIGHT WHEELCHAIR

### PRODUCT FEATURES



#### FOLDABLE BACKREST

Press on the lever to release and fold back rest downwards for both sides



#### FLIP UP ARM REST

Press button to release lock and flip up arm rest



#### DETACHABLE FOOTREST

Pull the lever to release lock and flip up arm rest



#### STATIONARY HAND BRAKE

Push up to unlock and down to lock

*For illustration purposes only*

### CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

## BOBBY ASTRO LIGHTWEIGHT WHEELCHAIR

### PRODUCT SPECIFICATIONS

Overall Length	102 cm / 40"
Overall Width	65.5 cm / 25.8"
Overall Height	92.2 cm / 36.3"
Folded Length	81 cm / 31.9"
Folded Width	38.5 cm / 15.2"
Folded Height	70.5 cm / 27.8"
Seat Depth	42 cm / 16.5"
Seat Width	46 cm / 18"
Seat Height	46.5 cm / 18.3"
Arm Rest Height	66.7 cm / 26.3"
Back Rest Height	42 cm / 16.5"
Front Wheel Diameter	15 cm or 6"
Back Wheel Diameter	50.8 cm / 20"
Safe Working Load	115 kg
Overall Weight	13.4 kg

Check our our full range of detachable wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

