

M187 RECLINING GERIATRIC CHAIR



For illustration purposes only

M187 RECLINING GERIATRIC CHAIR

PRODUCT OVERVIEW




For illustration purposes only

Rainbow Care's Geriatric Chairs are equipped with large padded cushion for added comfort. It comes with a detachable table for convenience and multi-functionality. Built to recline from 90° to 130°, pull the knob to recline.

Designed according to ergonomic principles for users' comfort, this Reclining Geriatric Chair is padded with Cushion Upholstery in its Back Rest, Arm Pads and Seat. It can be conveniently adjusted to meet individuals' needs. Its Versatility puts users and caretakers mind at ease.

M187 RECLINING GERIATRIC CHAIR

PRODUCT FEATURES

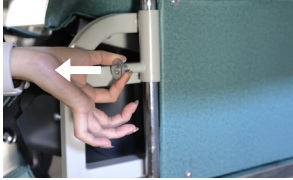

ADJUSTABLE ARM REST

Pull pin to adjust the height of the arm rest.



FOLDABLE SIDE TABLES

Turn up and push in to lock side tables.



RECLINING LEVER

Pull the lever back while person is sitting for recline.



LOCKABLE WHEELS

Push down inner part to unlock and outer to lock.

For illustration purposes only

CARE INSTRUCTIONS

- Ensure the wheels are locked before getting in and out of the Geriatric Chair.
- Use the safety belt to prevent user from sliding out of the chair
- Pay attention to the shift of the Center of Gravity when on elevated grounds
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the Geriatric Chair
- Do not exceed the maximum load

PRODUCT SPECIFICATIONS

M187 RECLINING GERIATRIC CHAIR

Overall Length	100.5 cm / 39.6"
Overall Width	82 cm / 32.3" (Without Table) 135.4 cm / 53.3" (With Table)
Overall Height	128.5 cm / 50.6"
Seat Depth	46 cm / 18.1"
Seat Width	54.5 cm / 21.5"
Seat Height	56 cm / 22"
Back Rest Height	81.5 cm / 32"
Table Overall Length	48 cm / 18.9"
Table Overall Width	25.5 cm / 10"
Table Overall Thickness	3.4 cm / 1.3"
Materials	Body: Steel Seat: PVC Leather
Safe Working Load	120 kg
Overall Weight	47.5 kg

Check out our full range of Reclining Geriatric Chairs. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

