

# KJW-312 DETACHABLE MOBILE COMMUNE



## KJW-312 DETACHABLE MOBILE COMMODE

### PRODUCT OVERVIEW



*For illustration purposes only*



*For illustration purposes only*

As a 3-in-1 Detachable Mobile Commode, a brilliant multi-tasker: a Commode Chair, Shower Chair, and as an Indoor Wheelchair.

- Foam Seat Cushions
- Lightweight Frame
- Detachable Bucket
- Detachable Foot Rest
- Flip Up Arm Rest
- 4 Wheels Lock

## DETACHABLE MOBILE COMMODE

### PRODUCT FEATURES



#### LEG REST

Pull the catch release to remove detachable leg rest



#### ARM REST

Press button to release lock and flip up arm rest



#### LOCKABLE WHEELS

Push up to unlock and down to lock

*For illustration purposes only*

### CARE INSTRUCTIONS

- When using mobility commode, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Commode.
- When getting in and out of the commode, please do not stand on the footplates.
- The footplates should be folded up beforehand, or removed.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the Commode.
- Do not exceed the maximum load.

PRODUCT SPECIFICATIONS

KJW-312 CHROME

|                      |                |
|----------------------|----------------|
| Overall Length       | 86 cm or 33.9" |
| Overall Width        | 54 cm or 21.3" |
| Overall Height       | 94 cm or 37"   |
| Seat Depth           | 42 cm or 16.5" |
| Seat Width           | 42 cm or 16.5" |
| Seat Height          | 50 cm or 19.7" |
| Front Wheel Diameter | 12.7 cm or 5"  |
| Back Wheel Diameter  | 12.7 cm or 5"  |
| Safe Working Load    | 100 kg         |
| Overall Weight       | 11 kg          |

Check out our full range of mobile commodes. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task, such as, such as transferring someone, are performed.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

