

PLUO ALUMINIUM COMMODE



PLUO ALUMINIUM MOBILE COMMODE

PRODUCT OVERVIEW



For illustration purposes only



For illustration purposes only

PLUO Aluminium Commode

The Pluo is an impressive 2-in-1 Multi-Functional Commode, featuring an aluminium frame, and ergonomic support for easy use.

- Flip Up Arm Rest
- Detachable Bucket
- Adjustable Foot Rest


PLUO ALUMINIUM MOBILE COMMODE

PRODUCT FEATURES



FOOT REST

Pull it out to use the foot rest. Make sure the front tube falls in the recess of the footrest support

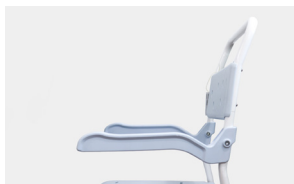

ARM REST

Turn it up to make room for passage. Turn it down to sit comfortably.



LOCKABLE WHEELS

Push up to unlock and down to lock

For illustration purposes only

CARE INSTRUCTIONS

- When using mobility commode, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the commode.
- When getting in and out of the commode, please do not stand on the footplates.
- The footplates should be folded up beforehand, or removed.
- Pay attention to the shift of the center of gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the commode.
- Do not exceed the maximum load.

PRODUCT SPECIFICATIONS

PLUO

Overall Length	59 - 83 cm or 23.2" - 32.6"
Overall Width	59.8 cm or 23.5"
Overall Height	106 cm or 41.7"
Seat Depth	45.5 cm or 17.9" (External) 37.5 cm or 14.8" (Internal)
Seat Width	49.5 cm or 19.5" (External) 19.5 cm or 7.7" (Internal)
Seat Height	51 cm or 20.1"
Back Rest Height	55 cm or 21.7"
Arm Rest Height	73 cm or 28.7"
Front Wheel Diameter	12.7 cm or 5"
Back Wheel Diameter	12.7 cm or 5"
Safe Working Load	150 kg
Overall Weight	9.7 kg

Check out our full range of mobile commodes. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain tasks, such as transferring someone, are performed.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

