

FT4213 STEEL STATIONARY COMMODE



FT4213 STEEL STATIONARY COMMODE



For illustration purposes only



For illustration purposes only

FT4213 Steel Stationary Commode

Made of lightweight frame and functions as a 2 in 1 commode where it is used as a commode chair and a shower chair. Foldable for storage.

FT4213 STEEL STATIONARY COMMODE

PRODUCT FEATURES



HEIGHT ADJUSTMENT

Press the button and pull / push to the level that you want



REMOVE BUCKET

Slide out to remove

CARE INSTRUCTIONS

- Place the commode on a flat and stable surface before use.
- Ensure all legs are adjusted to the same height and securely locked before sitting.
- Check that all rubber tips are fitted securely and are not excessively worn.
- Ensure the seat, armrests and bucket are properly positioned and secure before use.
- Do not exceed the safe working load stated for the product.
- When transferring to or from the commode, ensure the chair remains stable and does not move.
- Empty and clean the commode bucket after use. Wash with mild soap and water and dry thoroughly before replacing.
- Clean the seat and frame regularly using a soft cloth and mild cleaning solution. Avoid abrasive cleaners that may damage the surface.
- Regularly inspect the frame, adjustment points, seat and other fittings for looseness, cracks, damage or wear.
- Stop using the commode if any part appears loose, damaged or unstable, and arrange for inspection or replacement before further use.

PRODUCT SPECIFICATIONS

FT4213

Overall Length	52 cm or 20.5"
Overall Width	52.5 cm or 20.7"
Overall Height	68.5 cm - 78.5 cm 26.9" - 30.9"
Seat Depth	38 cm or 14.9"
Seat Width	42 cm or 16.5"
Seat Height	34.5 cm - 44.5 cm 13.6" - 17.5"
Safe Working Load	80 kg
Overall Weight	4.5 kg

Check our our full range of stationary commodes. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

