

# ECL X2T-46 ECLIPS DETACHABLE WHEELCHAIR

4



*For illustration purposes only*

**VERMEIREN**

## ECL X2T-46 ECLIPS DETACHABLE WHEELCHAIR

### PRODUCT OVERVIEW



### ECL X2T-46 Eclips Detachable Wheelchair



- Multi-function wheelchair with detachable foot rests
- Breathable woven nylon seat
- Lightweight aluminium frame

# DETACHABLE WHEELCHAIR

## PRODUCT FEATURES



### FOLDABLE BACKREST

Press on the lever to release and fold back rest downwards for both sides



### DETACHABLE FOOTREST

Pull the lever to release lock and flip up arm rest



### STATIONARY HAND BRAKE

Push up to unlock and down to lock

*For illustration purposes only*

## CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

## ECL X2T-46 ECLIPS DETACHABLE WHEELCHAIR

### PRODUCT SPECIFICATIONS

Overall Length	100 cm / 39.4"
Overall Width (Wheel to Wheel)	64 cm / 25.2"
Overall Height	92 cm / 36.2"
Folded Length	78 cm / 30.7"
Folded Width	35 cm / 13.8"
Folded Height	70 cm / 27.6"
Seat Depth	41 cm / 16.1"
Seat Width	46 cm / 18.1"
Seat Height	48 cm / 18.9"
Arm Rest Height (Floor to Arm Rest)	68.5 cm / 27"
Back Rest Height	44 cm / 17.3"
Front Wheel Diameter	15 cm / 5.9"
Back Wheel Diameter	60 cm / 24"
Safe Working Load	115 kg
Overall Weight	15 kg

Check our our full range of detachable wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

