FT6802 AIRPLANE WHEELCHAIR



FT6802 AIRPLANE WHEELCHAIR

PRODUCT OVERVIEW





Lightweight Wheelchair

- · Easy to fold and operate
- Multi-function wheelchair with fold-up foot rests for easier patient transfer.
- · Foldable frame for easy storage
- · Ventilated and cushioned seat upholstery
- · Aluminium lightweight frame
- · Pull-Trolley attachment for easy mobility



LIGHTWEIGHT WHEELCHAIR

PRODUCT FEATURES





FLIP UP ARM REST

Flip up arm rest by pushing up





HANDLE BRAKES

When brakes are engaged, press the red trigger upward to release and move the wheelchair.





FLIP UP FOOT REST

Hold the seat frame and backrest tube. Gently pull inward to close.

For illustration purposes only

CARE INSTRUCTIONS

- When using the wheelchair, please pay attention to people and/or items around to avoid collision.
- · Lock the wheels before getting in and out of the Wheelchair.
- · When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- · Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- · Do not exceed the maximum safe working load.

FT6802 AIRPLANE WHEELCHAIR

PRODUCT SPECIFICATIONS

Overall Length 88 cm / 34.6"

Overall Width (Wheel to Wheel) 51.5 cm / 20.3"

Overall Height 93 cm / 36.6"

Folded Length 54.5 cm / 21.5"

Folded Width (Wheel to Wheel) 43 cm / 16.9"

Folded Height 71 cm / 28"

Seat Depth 36 cm / 14.2"

Seat Width 40.5 cm / 16"

Seat Height (Floor to Seat Base) 51 cm / 20.1"

Arm Rest Height (Floor to Arm Rest) 73 cm / 28.7"

Back Rest Height 34 cm / 13.4"

Front Wheel Diameter 15 cm / 6"

Back Wheel Diameter 30.5 cm / 12"

Safe Working Load 90 kg

Overall Weight 7.6 kg

Check our our full range of detachable wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.



© 6223 0904 • 6931 7425 © 8501 1878 ≅ sales@rainbowcare.com.sg ⊕ rainbowcare.com.sg @ arainbowcare © rainbowcaresg ♀ Blk 3007 Ubi Road 1 #02-410 Singapore 408701