

HBL-40 RECLINING WHEELCHAIR



HBL-40 RECLINING WHEELCHAIR

PRODUCT OVERVIEW




KY954 Reclining Wheelchair

- Multi-function wheelchair with detachable foot rests and arm rests for easier patient transfer.
- Sized for children with a higher back rest and a seat block to prevent the child from sliding off the wheelchair while providing a good support.
- Breathable woven nylon seat
- Lightweight aluminium frame

RECLINING WHEELCHAIR

PRODUCT FEATURES

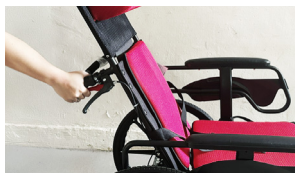

RECLINING BACKREST

Squeeze on the red lever to lower the backrest



FLIP-AWAY FOOTREST

Pull the lever to release the lock and remove accordingly



STATIONARY HAND BRAKE

Push up to unlock and down to lock



HEIGHT ADJUSTABLE ARM RESTS

Pull knob to unlock and shift the armrest into place.

For illustration purposes only

CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

HBL-40 RECLINING WHEELCHAIR

PRODUCT SPECIFICATIONS

Overall Length	120 cm or 47.2"
Overall Width (Wheel to Wheel)	70 cm or 27.6"
Overall Height	120 cm or 47.2" (With headrest) 90 cm or 35.4" (Without headrest)
Folded Length	98 cm or 38.6"
Folded Width	37 cm or 14.6"
Seat Depth	40 cm or 15.7"
Seat Width	45 cm or 17.7"
Seat Height	48 cm or 18.9"
Reclining Angle	Up to 180°
Arm Rest Height (Floor to Arm Rest)	65 - 75 cm or 25.6 - 29.5 "
Back Rest Height	43 cm or 16.9 " (without headrest) 84 cm or 33.1" (with headrest)
Front Wheel Diameter	18 cm or 7"
Back Wheel Diameter	53 cm or 21"
Safe Working Load	100 kg
Overall Weight	16 kg

Check our our full range of reclining wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

